

<Hospital_Name>

<Users_Name>

<Department>

<Customer_Address>

<Zip_Code> <City>

<Country_Name>

<Reference: 97434080-FA>

7 July 2025

Urgent Field Safety Notice - Urgent Medical Device Recall Carotid WALLSTENT™ Monorail Endoprosthesis

Dear Materials Manager and/or Healthcare Provider,

Boston Scientific is initiating the removal of certain batches of Carotid WALLSTENT™ Monorail Endoprosthesis (Carotid WALLSTENT) devices due to an increase in complaints received in which physicians described encountering greater than anticipated resistance when attempting to withdraw/retract the stent delivery system (SDS) from the guidewire or embolic protection device (EPD), after successful stent deployment.

The affected device batches, shown in Attachment 1, were manufactured with equipment settings that led to an unintended change resulting in the inner guidewire lumen being narrower than the specified dimensions. This change in the guidewire lumen dimension causes restricted guidewire movement in the affected batches. Boston Scientific has corrected the manufacturing equipment.

Device Description

The Carotid WALLSTENT consists of a stent and delivery system. The role of the stent is to provide a patent lumen so that the flow of blood into the carotid artery is restored. The function of the delivery catheter is to advance the non-deployed stent over a guidewire to the desired lesion location and deploy the stent.

Clinical Impact

Due to the guidewire lumen restriction, the inability to withdraw the SDS results in the need to remove the SDS and EPD as a system while the EPD is still deployed. If the EPD contains debris, this may be dislodged and spill into the bloodstream with consequent embolization into the cerebral vasculature. Although not observed in any of the reported complaints to date, the most serious potential adverse health consequence would be a life-threatening cerebrovascular accident (CVA)/stroke.

Additionally, there may be difficulty in retracting the deployed EPD through the carotid artery and Carotid WALLSTENT which may result in vessel injury, vessel spasm or stent disruption/damage necessitating additional endovascular or surgical interventions.

The most common adverse health consequence observed from this issue is a delay in the procedure beyond the expected timeframe. This may occur as the SDS and guidewire/EPD system is removed and access through the stented carotid artery is lost, potentially requiring replacement of guidewire access through the lesion to complete the procedure. If guidewire access through the Carotid WALLSTENT is required, there is a possibility of the guidewire not traversing through the true lumen of the stent which may result in vessel injury or stent disruption. This may necessitate additional endovascular or surgical interventions.

There is no need for any further specific patient management related to the issue described above if the Carotid WALLSTENT was successfully utilized and the procedure has been completed. Standard post-procedure care is appropriate.

Our records indicate that your facility has received some of the concerned product. **The table below (Attachment 1) provides a complete list of all affected products**, including Product Description, Material Number (UPN), GTIN and Lot/Batch numbers and expiry date. Please note that **only the devices listed below are affected. No other Boston Scientific product is involved in this Field Safety Notice.** **Further distribution or use of any remaining product affected by this action should cease immediately.**

INSTRUCTIONS:

1- **Please immediately discontinue use of the Boston Scientific product reported in the list and remove all of the affected units from your inventory**, regardless of where these units are stored in your facility. Segregate the units in a secure place, pending return to Boston Scientific.

2- **Please complete the attached Verification Form** even if you do not have any product to return.

3- **When completed, please return the Verification Form to your local Boston Scientific office** for the attention of <Customer_Service_Fax_Number> on or before **30 July 2025**.

4- **If you have products to return**, please package them in an appropriate shipping box. **After receipt of the Verification Form, Boston Scientific will contact you to arrange return.**

5- Please pass this notice to any health professional from your organization that needs to be aware and to any organization where the potentially affected devices have been transferred (if appropriate). Please provide Boston Scientific with details of any affected devices that have been transferred to other organizations (if appropriate).

Your Competent Authority is being notified of this Field Safety Notice.

We regret any inconvenience that this action may cause, and we appreciate your understanding as we act to ensure patient safety and customer satisfaction.

If you have any questions or would like assistance with this Field Safety Notice, please contact your local Sales Representative.

Yours sincerely,



Scott Heineman
Vice President, Global Quality
Boston Scientific Corporation

Attachment: Verification Form

Attachment 1- Affected Device List

Material UPN	Material Description	GTIN	Batch	Expiration Date Range
H7493915010240	CAROTID WALLSTENT MONORAIL 10.0-24	08714729781097	35277608, 35315983, 35315984, 35315985, 35380314, 35452835, 35452836, 35514614, 35551688, 35551689, 35664523, 35664524, 35750157, 35750158, 35750159, 35750400, 35879409, 35879410, 35921915, 35922951, 35926817, 35926818, 35942366, 36056965, 36058841, 36058842, 36058843, 36096638, 36121883, 36138618, 36138619, 36138801, 36138802, 36138803, 36159671, 36195057, 36195058, 36195059, 36195360, 36205167, 36209423, 36214805, 36214806, 36256192, 36322725, 36348207, 36348208, 36348209, 36498256, 36588496, 36607566, 36711321	14-May-2028 thru 8-Dec-2028
H7493915010310	CAROTID WALLSTENT MONORAIL 10.0-31	08714729781110	35294379, 35294540, 35295922, 35330503, 35330504, 35336132, 35347263, 35382292, 35383351, 35383352, 35453002, 35461487, 35461488, 35528579, 35529200, 35628964, 35628965, 35628966, 35641995, 35662047, 35664484, 35664485, 35750401, 35750402, 35750403, 35750404, 35750405, 35750406, 35750407, 35890012, 35893584, 35893585, 35894358, 35894359, 35938922, 35938923, 35938924, 35938925, 35938926, 36002641, 36002642, 36063291, 36063292, 36105446, 36133161, 36134808, 36144148, 36148535, 36154091, 36154092, 36154093, 36154094, 36154095, 36169871, 36169872, 36197809, 36197810, 36197811, 36197812, 36197813, 36200118, 36202762, 36204350, 36212466, 36212467, 36212468, 36256193, 36256194, 36256195, 36357711, 36357712, 36368143, 36368144, 36368145, 36516036, 36603362, 36603363, 36642784, 36683543, 36688775, 36811476, 36811477, 36818953	16-May-2028 thru 22-Dec-2028
H749391506220	CAROTID WALLSTENT MONORAIL 6.0-22	08714729781158	35582211, 35750408, 35947814, 35947815, 35950476, 36009906, 36292862, 36397124, 36652844	9-July-2028 thru 30-Nov-2028
H749391508210	CAROTID WALLSTENT MONORAIL 8.0-21	08714729781172	35343574, 35438658, 35487375, 35567637, 35652700, 35656388, 35750409, 35757024, 35873346, 35873347, 35901381, 35901382, 35999854, 35999855, 35999856, 36246587, 36246588, 36246589, 36292863, 36306518, 36376412, 36410418, 36410419, 36412851, 36479267, 36565560, 36636529, 36724350, 36777341	23-May-2028 thru 16-Dec-2028
H749391508290	CAROTID WALLSTENT MONORAIL 8.0-29	08714729781196	35339488, 35375416, 35375417, 35462381, 35529285, 35529286, 35617101, 35617102, 35618276, 35662042, 35662043, 35749222, 35750156, 35750410, 35750411, 35750412, 35750413, 35771618, 35944023, 35944024, 35946987, 35946988, 35950477, 35950478, 35950479, 35951080, 35951081, 35951082, 35951083, 35961400, 35978120, 36320917, 36320918, 36320919, 36321560, 36322143, 36458848, 36458849, 36458850, 36458851, 36458852, 36465397, 36465398, 36465399, 36505552, 36505553, 36607622, 36683540, 36792075	22-May-2028 thru 18-Dec-2028
H965SCH647010	CAROTID WALLSTENT MONORAIL 6.0-22	08714729263869	35347272, 35347273, 35347274, 35432322, 35432323, 35432324, 35432325, 35435587, 35496538, 35496539, 35510448, 35510449, 35549114, 35558691, 35558692, 35623470, 35623471, 35623472, 35641992, 35757022, 35891995, 35891996, 35903878, 35903879, 35904080, 35904081, 35911104, 35911106, 35926007, 35926017, 35990353, 35990354, 36009907, 36012786, 36012787, 36012788, 36012789, 36222681, 36222683, 36241106, 36241107, 36241108, 36292865, 36292866, 36292867,	23-May-2028 thru 17-Dec-2028

			36292868, 36292869, 36292870, 36297506, 36297507, 36348203, 36348204, 36357034, 36357035, 36357037, 36402896, 36402897, 36469383, 36469384, 36469385, 36493671, 36515107, 36609942, 36609943, 36609944, 36717345, 36722322, 36769772, 36769773, 36787067	
H965SCH647070	CAROTID WALLSTENT MONORAIL 8.0-21	08714729263876	35240183, 35240184, 35240185, 35241830, 35241831, 35242395, 35244198, 35302653, 35302654, 35345323, 35345324, 35345325, 35353957, 35353958, 35353959, 35355526, 35355527, 35355528, 35355529, 35355530, 35355531, 35355532, 35355533, 35355534, 35355535, 35388600, 35388601, 35388602, 35388603, 35388604, 35388605, 35390839, 35432326, 35435588, 35435589, 35435590, 35435591, 35435592, 35435593, 35435594, 35435595, 35435596, 35435597, 35435598, 35435599, 35435640, 35435641, 35435642, 35438652, 35438653, 35438654, 35438659, 35448174, 35448175, 35448261, 35448262, 35448263, 35448264, 35487376, 35487377, 35489534, 35489535, 35489536, 35494158, 35494159, 35495360, 35495361, 35495362, 35495363, 35497060, 35497061, 35497062, 35507545, 35507546, 35514618, 35548744, 35549113, 35553916, 35553917, 35553918, 35553919, 35556850, 35556851, 35556859, 35556860, 35565001, 35565003, 35567636, 35567638, 35574827, 35580272, 35580273, 35580274, 35580275, 35580276, 35582209, 35582210, 35584963, 35588884, 35588885, 35588886, 35610418, 35610419, 35618278, 35618279, 35620887, 35620888, 35622207, 35643329, 35646460, 35646461, 35646462, 35646465, 35646466, 35646467, 35646469, 35646470, 35646471, 35646472, 35646473, 35652701, 35652702, 35652703, 35656390, 35771619, 35771720, 35771721, 35771722, 35771723, 35771724, 35774933, 35774935, 35912369, 35912370, 35912371, 35912372, 35913736, 35934661, 35934662, 35934663, 35934664, 35936385, 35936386, 35936388, 35937854, 35937855, 35937961, 35937962, 35938832, 35938833, 35944025, 35944026, 35944027, 35944028, 35946402, 35946403, 35952677, 35952678, 35952679, 35952740, 35960613, 35960614, 35960615, 35960616, 35960617, 35960618, 35960619, 35970056, 35970057, 35970482, 35970483, 35972008, 35972009, 35972010, 35972011, 35974039, 35974900, 35974902, 35974903, 35978121, 35978122, 35979584, 35979585, 35985615, 35985616, 35988023, 35989721, 35999859, 35999920, 35999976, 35999977, 36058848, 36058849, 36058850, 36058851, 36058852, 36058853, 36058854, 36066439, 36066960, 36067793, 36067794, 36067795, 36067796, 36067797, 36067798, 36067799, 36068080, 36072053, 36072054, 36091095, 36091096, 36091097, 36093208, 36093209, 36093740, 36093741, 36094572, 36094573, 36099091, 36099092, 36099093, 36103432, 36103433, 36230819, 36231140, 36233435, 36233436, 36233437, 36233438, 36241642, 36241643, 36241660, 36241661, 36243288, 36243289, 36243290, 36246584, 36246585, 36246586, 36246590, 36246591, 36246592, 36246593, 36246594, 36247809, 36247810, 36247811, 36247812, 36251034, 36251035, 36251036, 36251037, 36251038, 36261365, 36261704, 36266101, 36266102, 36266103, 36266104, 36278239, 36278520, 36278521, 36278522, 36278523, 36278524, 36278525, 36281013, 36281014, 36281015, 36281016, 36281017, 36281018, 36281019, 36281520, 36281521, 36281522, 36285859, 36286700, 36286701, 36286702, 36286703, 36286704, 36286705, 36286706, 36286707, 36286708, 36287360, 36287361, 36292139, 36292861, 36297762, 36301606, 36301607, 36301608, 36301609, 36301610, 36301611, 36301612, 36301613, 36301614, 36301615, 36306516, 36306517, 36352994, 36352996, 36352997, 36352999, 36353020, 36353022, 36353023,	8-May-2028 thru 17-Dec-2028

			36353025, 36353026, 36353028, 36357708, 36357709, 36357710, 36362685, 36362686, 36362687, 36362688, 36362689, 36362690, 36362691, 36362692, 36362693, 36362694, 36368150, 36368151, 36368152, 36368153, 36368154, 36372913, 36372914, 36372915, 36372916, 36372917, 36372918, 36372919, 36373140, 36373141, 36373142, 36376410, 36376411, 36402898, 36402899, 36403020, 36408269, 36408270, 36408271, 36408272, 36408273, 36408274, 36408275, 36408276, 36408277, 36408278, 36408279, 36410417, 36412854, 36423241, 36423242, 36469386, 36469387, 36469388, 36469389, 36475479, 36475580, 36475581, 36479261, 36479262, 36479263, 36479264, 36479265, 36479266, 36479268, 36480347, 36501126, 36515108, 36523367, 36525434, 36525435, 36525437, 36525438, 36565561, 36609945, 36609946, 36609947, 36609948, 36609950, 36609951, 36609952, 36616128, 36616129, 36616130, 36616131, 36616132, 36645415, 36645416, 36668237, 36668238, 36668239, 36668460, 36668461, 36668462, 36669215, 36672359, 36688777, 36712752, 36717346, 36717347, 36717348, 36719378, 36721745, 36721746, 36722177, 36722179, 36722320, 36724346, 36724347, 36724348, 36724349, 36728508, 36728509, 36729021, 36729022, 36729023, 36732219, 36732360, 36732361, 36732362, 36732974, 36732976, 36732977, 36743995, 36743996, 36743997, 36773856, 36773857, 36773858, 36782024, 36784204	
H965SCH647080	CAROTID WALLSTENT MONORAIL 8.0-29	08714729263883	35243220, 35244199, 35244520, 35244521, 35247274, 35247275, 35247276, 35247277, 35247278, 35247279, 35247540, 35247541, 35248292, 35248293, 35248294, 35252806, 35252808, 35252809, 35302655, 35302656, 35302657, 35302658, 35304581, 35305305, 35305306, 35307512, 35307513, 35307514, 35339484, 35339485, 35339486, 35339487, 35357498, 35357499, 35357740, 35361258, 35361259, 35361460, 35361461, 35361462, 35361463, 35362807, 35362808, 35365037, 35365038, 35365039, 35365800, 35365801, 35365802, 35365803, 35365804, 35365805, 35367555, 35370836, 35370837, 35370838, 35370839, 35371180, 35371181, 35371182, 35371183, 35375418, 35375419, 35388606, 35388607, 35388608, 35388609, 35388610, 35388611, 35391026, 35391027, 35391028, 35391029, 35391030, 35393507, 35393508, 35393509, 35393510, 35393511, 35393512, 35394158, 35394159, 35395620, 35395621, 35395622, 35395623, 35395624, 35395625, 35395626, 35395627, 35397442, 35397443, 35397444, 35397445, 35402251, 35402254, 35402257, 35402280, 35402283, 35402284, 35406000, 35406001, 35448176, 35448177, 35450110, 35450111, 35450112, 35450113, 35456213, 35456215, 35456216, 35456217, 35457525, 35457526, 35458769, 35461739, 35461741, 35461742, 35462380, 35462382, 35462383, 35465421, 35465422, 35465423, 35465424, 35468042, 35468043, 35468044, 35468045, 35468046, 35468047, 35468048, 35468049, 35468050, 35468051, 35471389, 35471390, 35471391, 35471392, 35471393, 35476347, 35476348, 35514612, 35518815, 35519420, 35520154, 35520155, 35520156, 35522424, 35522425, 35522699, 35522780, 35522781, 35522782, 35522977, 35522978, 35522979, 35523220, 35523221, 35523222, 35525808, 35525809, 35525810, 35525811, 35526986, 35526987, 35526988, 35528577, 35528578, 35529284, 35529287, 35529629, 35529630, 35529631, 35529632, 35531670, 35531671, 35531672, 35541480, 35541481, 35541482, 35541483, 35542643, 35562259, 35562520, 35565634, 35565635, 35565636, 35585834, 35585835, 35585836, 35585838, 35589525, 35589526, 35590013, 35590014, 35611923, 35611924, 35612128, 35612129,	9-May- 2028 thru 22-Dec- 2028

			35612130, 35612131, 35612132, 35612143, 35612144, 35613461, 35613462, 35613463, 35613464, 35613465, 35616796, 35616797, 35616798, 35616799, 35617100, 35618178, 35618179, 35618277, 35620886, 35622733, 35622738, 35622760, 35622761, 35636552, 35636553, 35652778, 35652779, 35653341, 35653342, 35653343, 35655507, 35655508, 35655510, 35655513, 35655514, 35656381, 35656383, 35656384, 35656386, 35656387, 35662005, 35662006, 35662040, 35662041, 35662044, 35662045, 35662046, 35749223, 35755824, 35774932, 35781375, 35781376, 35781377, 35781378, 35781379, 35781880, 35866470, 35866471, 35867497, 35868573, 35868574, 35870253, 35871472, 35871476, 35872080, 35872081, 35873025, 35873026, 35876804, 35876805, 35879407, 35879408, 35881807, 35881808, 35887451, 35887452, 35887453, 35887454, 35891991, 35891992, 35891993, 35891994, 35896872, 35896873, 35899097, 35903147, 35903148, 35903149, 35903150, 35912756, 35912757, 35925347, 35926819, 35928104, 35928105, 35929567, 35929568, 35932458, 35932459, 35979594, 35979595, 35983716, 35988028, 35988445, 35988447, 35989162, 35989177, 35989178, 35989279, 35989720, 35989945, 35989946, 35994087, 35994089, 35994090, 35994091, 35995228, 35995229, 35995230, 35995231, 35997085, 35997978, 35997979, 35998240, 35998241, 35998242, 36006192, 36006193, 36006194, 36006195, 36006196, 36006197, 36006198, 36012790, 36012791, 36012792, 36012793, 36018044, 36018045, 36018046, 36018047, 36026734, 36026735, 36026736, 36026737, 36026738, 36026739, 36026960, 36026961, 36026963, 36026964, 36032712, 36032715, 36032717, 36032719, 36032981, 36036491, 36036492, 36036493, 36036494, 36036495, 36036496, 36038161, 36038162, 36038163, 36040721, 36040722, 36040723, 36040724, 36040725, 36040726, 36040727, 36040728, 36043491, 36043492, 36043493, 36043494, 36048928, 36048929, 36048930, 36048931, 36048932, 36048933, 36048934, 36048935, 36048936, 36048937, 36048938, 36048939, 36055143, 36055144, 36068081, 36068082, 36068083, 36068084, 36068085, 36068086, 36078464, 36078465, 36078466, 36078467, 36078468, 36087121, 36087122, 36087123, 36087124, 36087125, 36087126, 36092012, 36092013, 36219930, 36219931, 36219932, 36219933, 36224134, 36224135, 36224136, 36227992, 36227993, 36227994, 36227995, 36228449, 36228450, 36228451, 36228753, 36228754, 36233439, 36233900, 36256580, 36256581, 36256582, 36256583, 36256584, 36256585, 36256586, 36256587, 36258193, 36258194, 36258195, 36258196, 36261366, 36261367, 36261368, 36261369, 36261370, 36261371, 36261373, 36261374, 36261376, 36266106, 36266107, 36266110, 36266111, 36266112, 36268719, 36269061, 36269062, 36270877, 36270878, 36270879, 36271660, 36271661, 36306519, 36306780, 36306781, 36306782, 36306783, 36306784, 36306785, 36306786, 36311913, 36311914, 36311915, 36311916, 36314781, 36314782, 36314783, 36314784, 36314785, 36314786, 36314787, 36314788, 36315939, 36316920, 36316921, 36316922, 36316923, 36318889, 36318890, 36318891, 36318892, 36318893, 36318894, 36318895, 36318896, 36318897, 36318898, 36320911, 36320912, 36320913, 36320914, 36320915, 36320916, 36322144, 36325066, 36325067, 36325068, 36327346, 36327347, 36327352, 36327358, 36327662, 36369519, 36369780, 36369781, 36378265, 36378266, 36378267, 36378268, 36378269, 36378270, 36378271, 36386880, 36386881, 36386882, 36386883, 36386884, 36386885, 36392438, 36392439, 36392520, 36392521, 36392522, 36396975, 36396976,
--	--	--	---

			36396977, 36396978, 36396979, 36397120, 36413022, 36413024, 36413025, 36413026, 36413027, 36413028, 36413029, 36413030, 36413031, 36413032, 36415159, 36416042, 36416043, 36416044, 36416334, 36416335, 36416336, 36416337, 36416338, 36416339, 36416420, 36422413, 36422414, 36422598, 36422599, 36423963, 36423964, 36423965, 36423995, 36423996, 36447747, 36447748, 36447749, 36448144, 36448145, 36448156, 36448157, 36449975, 36449976, 36449977, 36453798, 36455025, 36455026, 36458843, 36458844, 36458845, 36458846, 36458847, 36480348, 36480349, 36480350, 36480351, 36485714, 36485715, 36485716, 36485717, 36485718, 36485719, 36485800, 36485801, 36485802, 36485803, 36485851, 36485852, 36491169, 36491170, 36491171, 36491172, 36491173, 36491174, 36493672, 36493673, 36493674, 36493675, 36493676, 36493677, 36498253, 36498254, 36498255, 36501125, 36508768, 36508771, 36508772, 36508773, 36508774, 36525439, 36529070, 36529071, 36529072, 36529073, 36529074, 36529075, 36529076, 36529077, 36529078, 36529079, 36535347, 36535348, 36535349, 36535352, 36535353, 36535354, 36535355, 36539304, 36539306, 36545277, 36545278, 36545279, 36545760, 36545761, 36548829, 36548830, 36552374, 36553653, 36553654, 36553655, 36554562, 36554563, 36554564, 36554565, 36554566, 36555138, 36560052, 36560053, 36560054, 36560055, 36560056, 36566126, 36566127, 36566128, 36566129, 36567621, 36571196, 36571197, 36571198, 36571199, 36571780, 36571781, 36572200, 36572201, 36572202, 36572203, 36572204, 36574800, 36574801, 36574802, 36575688, 36575689, 36575690, 36577074, 36577075, 36578325, 36578327, 36578328, 36582969, 36586443, 36618329, 36620780, 36620781, 36620782, 36620783, 36620784, 36620785, 36620786, 36620787, 36620788, 36625778, 36625779, 36626060, 36626061, 36626062, 36626063, 36626064, 36626065, 36626066, 36626067, 36630781, 36630782, 36630783, 36630784, 36630785, 36630786, 36630787, 36630788, 36630789, 36630790, 36645417, 36645418, 36645419, 36673680, 36673681, 36673682, 36673683, 36675542, 36675543, 36675545, 36675546, 36675547, 36675549, 36675550, 36675551, 36675552, 36678442, 36678443, 36678444, 36678445, 36678446, 36678447, 36678448, 36683541, 36688432, 36688433, 36688434, 36712751, 36717343, 36738231, 36738232, 36739201, 36739203, 36739206, 36739207, 36739208, 36739209, 36741080, 36741081, 36742784, 36742785, 36742786, 36742787, 36742788, 36742789, 36748006, 36748007, 36748008, 36749662, 36749663, 36761491, 36761492, 36765445, 36765446, 36767807, 36784206, 36785292, 36785384, 36785385, 36787969, 36787970, 36787971, 36788799, 36792076, 36792077, 36793502, 36793503, 36793678, 36793679, 36795349, 36795350, 36795351, 36799042, 36799043, 36801653, 36803154, 36803155, 36817363, 36818518, 36818519	
H965SCH647090	CAROTID WALLSTENT MONORAIL 8.0-36	08714729263890	35284214, 35284216, 35284217, 35296025, 35296026, 35296027, 35307517, 35309535, 35309536, 35313433, 35313434, 35313435, 35343571, 35376080, 35376081, 35376082, 35376083, 35395628, 35395629, 35395630, 35406003, 35452838, 35471394, 35471395, 35471396, 35471397, 35471398, 35476350, 35476351, 35476352, 35476819, 35476960, 35514613, 35541484, 35541485, 35541486, 35541487, 35542644, 35542645, 35542646, 35542647, 35588888, 35588889, 35588890, 35594003, 35600380, 35600381, 35600716, 35600717, 35600718, 35663304, 35663305, 35663307, 35663958, 35663959, 35664480, 35664482, 35667604, 35749225, 35749227, 35749228, 35778247, 35778248, 35778249, 35778250,	15-May- 2028 thru 22-Dec- 2028

			35893582, 35893583, 35928594, 35928595, 35950473, 35950474, 35950475, 35964009, 35964010, 35972012, 35980330, 35980331, 36007692, 36007693, 36007694, 36007695, 36007696, 36018041, 36018042, 36033496, 36033497, 36033498, 36033499, 36033880, 36043497, 36043498, 36043499, 36043780, 36055146, 36055147, 36055785, 36056677, 36078428, 36097193, 36116118, 36116119, 36116120, 36220947, 36322723, 36322724, 36328322, 36328323, 36328324, 36341859, 36342380, 36342381, 36342382, 36342383, 36342384, 36342385, 36342386, 36342387, 36342388, 36347198, 36347199, 36347220, 36347221, 36347222, 36347223, 36347224, 36347225, 36347226, 36357714, 36357715, 36378272, 36378273, 36465500, 36465501, 36469380, 36469381, 36469382, 36475477, 36475478, 36508775, 36508776, 36513257, 36513258, 36513259, 36513760, 36518682, 36518683, 36553656, 36553657, 36553658, 36553659, 36554560, 36563910, 36582970, 36582971, 36582972, 36582973, 36582974, 36595826, 36595828, 36635529, 36635530, 36635531, 36646050, 36646051, 36679165, 36688435, 36712758, 36717344, 36795885, 36817573	
H965SCH647120	CAROTID WALLSTENT MONORAIL 10.0-24	08714729263906	35258269, 35258270, 35258273, 35258274, 35260228, 35260229, 35265522, 35265523, 35266396, 35266397, 35266398, 35266399, 35267871, 35267872, 35272291, 35272292, 35273687, 35273688, 35273689, 35274001, 35274002, 35274003, 35274004, 35276183, 35276184, 35276185, 35276186, 35276187, 35277609, 35278162, 35278163, 35278164, 35278165, 35278167, 35281328, 35281329, 35289305, 35306610, 35306611, 35306612, 35310693, 35310694, 35310695, 35310696, 35315986, 35315987, 35315988, 35320538, 35320539, 35324563, 35324564, 35324565, 35324566, 35324567, 35324568, 35324569, 35376084, 35376085, 35376087, 35380104, 35380105, 35380315, 35380316, 35381858, 35381859, 35381980, 35381981, 35381982, 35381983, 35381984, 35395631, 35395632, 35395633, 35406004, 35406005, 35406007, 35406008, 35406009, 35406010, 35409844, 35409845, 35409847, 35409848, 35409849, 35409851, 35409852, 35409853, 35416153, 35416154, 35452819, 35453000, 35461486, 35480117, 35480118, 35480946, 35480947, 35480948, 35480949, 35484493, 35484494, 35487378, 35487379, 35501516, 35501517, 35514615, 35514616, 35514617, 35515702, 35517161, 35519704, 35545687, 35545688, 35545689, 35545690, 35545691, 35546819, 35547736, 35547737, 35551690, 35551691, 35551692, 35565638, 35565640, 35565641, 35565642, 35565643, 35569825, 35569826, 35569827, 35594002, 35626901, 35665247, 35667601, 35667602, 35671453, 35671454, 35671455, 35671456, 35671457, 35673280, 35890013, 35890014, 35895660, 35895661, 35901388, 35901389, 35921353, 35921354, 35921355, 35921356, 35942368, 35998696, 35998697, 35998698, 36001798, 36001799, 36002640, 36009902, 36009903, 36009904, 36009905, 36012794, 36043495, 36043496, 36056678, 36056679, 36056960, 36056961, 36056964, 36058844, 36058845, 36058846, 36058847, 36063287, 36063288, 36063290, 36068087, 36068088, 36068089, 36068090, 36095403, 36095404, 36095405, 36095406, 36096639, 36101301, 36101302, 36101699, 36102320, 36102321, 36102324, 36102325, 36103434, 36103435, 36103436, 36105447, 36105448, 36111547, 36111548, 36111549, 36111551, 36111552, 36112953, 36112954, 36112955, 36112956, 36112957, 36121884, 36138616, 36138617, 36138800, 36138804, 36138805, 36159666, 36159667, 36159668, 36159669, 36159670, 36159672, 36159673, 36164841, 36164842, 36164843, 36169145, 36169146, 36169147, 36169148, 36169867, 36169868, 36169869,	12-May- 2028 thru 24-Dec- 2028

			36173875, 36173876, 36179702, 36179703, 36180136, 36193169, 36193170, 36193171, 36193172, 36193173, 36195518, 36195519, 36195580, 36195581, 36195582, 36205168, 36205169, 36208940, 36208941, 36209024, 36209025, 36209424, 36224696, 36224697, 36234046, 36234047, 36234048, 36498257, 36498258, 36498259, 36501127, 36586444, 36586445, 36586446, 36586447, 36588495, 36588497, 36591213, 36591214, 36593068, 36593069, 36607567, 36640275, 36640276, 36640277, 36679166, 36683542, 36688773, 36688774, 36698998, 36698999, 36711322, 36711323, 36787078, 36789908, 36789909, 36796820, 36800990, 36803152, 36834285	
H965SCH647130	CAROTID WALLSTENT MONORAIL 10.0-31	08714729263913	35258275, 35258276, 35258277, 35258278, 35258279, 35260230, 35260231, 35261618, 35261619, 35264188, 35264189, 35264190, 35264191, 35264192, 35266488, 35266489, 35268647, 35268648, 35278746, 35281326, 35281327, 35285493, 35285494, 35285495, 35285496, 35285497, 35285498, 35285499, 35285920, 35286196, 35286197, 35286198, 35286199, 35286540, 35287317, 35287319, 35287401, 35289015, 35289303, 35293059, 35293160, 35293161, 35293162, 35293163, 35293812, 35293813, 35293815, 35293816, 35320520, 35320521, 35320522, 35320523, 35320525, 35325953, 35325954, 35325955, 35329088, 35329090, 35329091, 35329092, 35329093, 35329094, 35329095, 35329096, 35329544, 35329545, 35329546, 35329547, 35329548, 35329549, 35329968, 35329969, 35329970, 35329971, 35330502, 35336128, 35336129, 35336130, 35336131, 35341680, 35353660, 35353661, 35353662, 35353663, 35353664, 35361464, 35361465, 35361466, 35361467, 35381985, 35382290, 35382291, 35383353, 35383354, 35383355, 35383356, 35383357, 35383358, 35383359, 35383620, 35383622, 35383623, 35386372, 35386373, 35386374, 35409855, 35409856, 35409857, 35409858, 35416156, 35416157, 35416158, 35416159, 35416460, 35416461, 35416462, 35416463, 35416464, 35416465, 35416466, 35416467, 35416468, 35416469, 35416470, 35422921, 35422928, 35422929, 35422930, 35424325, 35424326, 35424327, 35424328, 35424329, 35424330, 35424331, 35424332, 35424333, 35424335, 35424336, 35424337, 35424338, 35424339, 35428634, 35428635, 35428636, 35428637, 35428638, 35428639, 35438650, 35438651, 35461489, 35462384, 35462385, 35462386, 35462387, 35462388, 35480950, 35480951, 35480952, 35480953, 35480955, 35480956, 35484496, 35484497, 35484498, 35501518, 35501519, 35501920, 35501921, 35501922, 35501923, 35525176, 35525177, 35525806, 35525807, 35530632, 35530633, 35530634, 35531679, 35531880, 35533597, 35533598, 35533599, 35534100, 35534101, 35534102, 35553908, 35553909, 35553910, 35553911, 35553937, 35553938, 35556848, 35556849, 35569824, 35570978, 35570979, 35571040, 35571538, 35571539, 35574379, 35574500, 35574501, 35574502, 35574503, 35578673, 35578674, 35578675, 35594949, 35623467, 35623468, 35626636, 35626637, 35626638, 35626639, 35626900, 35627523, 35627524, 35630022, 35630023, 35632452, 35632454, 35632455, 35632638, 35632639, 35633487, 35633492, 35633901, 35633903, 35636548, 35636549, 35636550, 35636551, 35667603, 35668453, 35668454, 35671444, 35671445, 35671447, 35671448, 35671449, 35673282, 35673283, 35677031, 35777662, 35777664, 35781899, 35782160, 35782161, 35782162, 35782163, 35782164, 35873344, 35873345, 35881810, 35881811, 35881812, 35881813, 35884601, 35884602, 35890010, 35901386, 35901387, 35904064, 35904065, 35942369, 35964011, 35964012, 35964013, 35964014, 35985601, 35985602, 36012795, 36012796, 36012797,	12-May- 2028 thru 25-Dec- 2028

			36012798, 36012799, 36012980, 36032984, 36068091, 36068092, 36105449, 36105450, 36105451, 36106593, 36106594, 36106595, 36106596, 36109872, 36109874, 36111550, 36116121, 36116122, 36116123, 36119507, 36119508, 36121880, 36121881, 36121882, 36121885, 36121886, 36124483, 36124484, 36130876, 36130877, 36130878, 36130879, 36131000, 36132932, 36132933, 36132934, 36132935, 36132936, 36132937, 36144149, 36144150, 36144151, 36144152, 36144153, 36144154, 36144155, 36144156, 36148525, 36148526, 36148823, 36148825, 36148827, 36148829, 36148830, 36154096, 36154097, 36154098, 36154099, 36154980, 36154981, 36154982, 36154983, 36154984, 36154985, 36154987, 36164844, 36164845, 36164846, 36164847, 36173877, 36173878, 36173879, 36174740, 36174741, 36174742, 36174743, 36174744, 36180137, 36180138, 36185918, 36185919, 36186400, 36186401, 36186402, 36186403, 36187901, 36187903, 36187904, 36187905, 36187906, 36194716, 36195042, 36200119, 36200760, 36202408, 36202409, 36205165, 36205166, 36212469, 36214814, 36214815, 36214816, 36219092, 36219093, 36219094, 36219095, 36219096, 36422415, 36422416, 36422417, 36516037, 36516038, 36516039, 36518684, 36518685, 36518686, 36518687, 36518688, 36518689, 36518690, 36518691, 36518693, 36520868, 36593070, 36593071, 36595822, 36595823, 36595824, 36595825, 36597571, 36597572, 36597573, 36597574, 36603361, 36603364, 36603365, 36603366, 36605707, 36606062, 36606063, 36606064, 36606065, 36606066, 36606067, 36606068, 36606069, 36640278, 36640279, 36641940, 36641941, 36642785, 36644287, 36644288, 36644752, 36644753, 36644754, 36664959, 36664987, 36664990, 36666077, 36666520, 36666522, 36666524, 36669032, 36669035, 36669036, 36669037, 36699060, 36699061, 36700303, 36712756, 36717599, 36717900, 36717901, 36717902, 36717903, 36717904, 36722328, 36722330, 36750320, 36750321, 36750322, 36750323, 36750324, 36752754, 36752755, 36752756, 36752758, 36752759, 36752900, 36766975, 36766976, 36766977, 36768310, 36768752, 36768753, 36773943, 36795889, 36795890, 36800991, 36800992, 36803691, 36804582, 36811474, 36811478, 36811479, 36812060, 36813561, 36813562, 36815619, 36815667, 36817342, 36844079	
H965SCH647140	CAROTID WALLSTENT MONORAIL 10.0-37	08714729263920	35272289, 35272290, 35278747, 35278748, 35293817, 35293818, 35293819, 35309703, 35309704, 35309705, 35309706, 35320537, 35329976, 35329978, 35329979, 35341681, 35341682, 35344559, 35345321, 35345329, 35347267, 35347268, 35347269, 35383621, 35386679, 35386780, 35386781, 35388276, 35388277, 35428800, 35428801, 35428802, 35428803, 35484499, 35489527, 35489530, 35489533, 35496534, 35496535, 35496536, 35525178, 35530635, 35560220, 35560221, 35560222, 35566231, 35625591, 35625592, 35641157, 35641460, 35677035, 35677036, 35677037, 35677038, 35682786, 35682787, 35682788, 35682789, 35682790, 35682791, 35778251, 35778252, 35987778, 35987779, 36066437, 36066438, 36072052, 36072851, 36072852, 36072854, 36105453, 36115283, 36115284, 36124482, 36132938, 36132939, 36133160, 36134809, 36154986, 36159663, 36164696, 36164697, 36164698, 36164699, 36179704, 36179705, 36179706, 36195056, 36202763, 36205164, 36213495, 36213496, 36213572, 36213573, 36213574, 36213575, 36498341, 36501128, 36501129, 36606591, 36652845, 36652846, 36773180, 36773181, 36773182, 36775992, 36775994, 36818514, 36818515	14-May- 2028 thru 22-Dec- 2028

M001719000	CAROTID WALLSTENT MR 6X22 5F 135CM	08714729781165	35438655, 35438656, 35438657, 35641993, 35641994, 36053998, 36053999, 36223361, 36223362, 36348206, 36523368, 36616127, 36777256	6-June- 2028 thru 16-Dec- 2028
M001719010	CAROTID WALLSTENT MR 8X21 5F 135CM	08714729781189	35252807, 35307515, 35307516, 35355536, 35450114, 35450115, 35504179, 35504200, 35504201, 35571041, 35583635, 35583636, 35583637, 36223363, 36223364, 36224129, 36224132, 36224133, 36230818, 36297515, 36297516, 36297518, 36297761, 36368141, 36368142, 36412855, 36475475, 36539307, 36616133, 36616134, 36739202	11-May- 2028 thru 11-Dec- 2028
M001719020	CAROTID WALLSTENT MR 8X29 5F 135CM	08714729781202	35252812, 35343568, 35343569, 35343570, 35343572, 35355537, 35379661, 35406002, 35476349, 35480115, 35480116, 35542648, 35560230, 35560231, 35629995, 35629996, 35629997, 35636546, 35636547, 35662002, 35662003, 35961401, 35961402, 35961403, 35961404, 35961405, 35970484, 35972013, 35977619, 36006440, 36007615, 36007616, 36007618, 36271662, 36271663, 36271664, 36271665, 36271666, 36275712, 36275713, 36275714, 36275715, 36275716, 36299150, 36299151, 36299152, 36299153, 36299154, 36306787, 36306788, 36397121, 36397122, 36453791, 36453792, 36453793, 36453795, 36475476, 36552375, 36552376, 36565457, 36617888, 36630791, 36633072, 36635526, 36767805, 36767806	11-May- 2028 thru 15-Dec- 2028
M001719030	CAROTID WALLSTENT MR 8X36 5F 135CM	08714729781226	35343573, 35347264, 35379662, 35476961, 35476962, 35545686, 35550309, 35550605, 35589532, 35664483, 35749224, 35980332, 35980333, 35997086, 36018043, 36078429, 36087098, 36087099, 36087540, 36096632, 36096633, 36347227, 36397123, 36416430, 36416431, 36563911, 36565458, 36565459, 36635532, 36635533	23-May- 2028 thru 27-Nov- 2028
M001719040	CAROTID WALLSTENT MR 10X24 5.9F 135CM	08714729781103	35281323, 35281324, 35281325, 35324570, 35324571, 35416155, 35484495, 35547738, 35623469, 36113886, 36113887, 36115281, 36115282, 36124259, 36124480, 36124481, 36169150, 36209425, 36498340, 36711739, 36712757, 36713869	15-May- 2028 thru 8-Dec- 2028
M001719050	CAROTID WALLSTENT MR 10X31 5.9F 135CM	08714729781127	35293814, 35295923, 35347265, 35347266, 35388278, 35485560, 35553939, 35641991, 35682792, 35939213, 35939214, 35939215, 35939216, 36065524, 36065525, 36065526, 36148529, 36148532, 36508770, 36606592, 36771557, 36817706	16-May- 2028 thru 22-Dec- 2028
M001719060	CAROTID WALLSTENT MR 10X37 5.9F 135CM	08714729781141	35310697, 35315981, 35315982, 35347270, 35388279, 35496537, 35566234, 35682793, 35978123, 36043781, 36105452, 36159664, 36159665, 36164840, 36498342	19-May- 2028 thru 10-Nov- 2028

<Sold_to> - <Hospital_Name> - <City> - <Country_Name>

Please Complete the form even if you do not have any affected product & send it to your Local Office:
 <Customer_Service_Fax_Number>

Verification Form – Urgent Medical Device Recall
Carotid WALLSTENT™ Monorail Endoprosthesis
97434080-FA

1. We acknowledge receipt of the Boston Scientific Field Safety Notice dated 7 July 2025.

2. **Boston Scientific records indicate you have received the following affected product** (*additionally please check inventory against complete list of affected product provided*)

Material N° (UPN)	Lot / Batch N°	Customer PO	Qty Sent	Qty to return (Units)

3. We confirm that all areas where affected product could be located have been checked.

4. **TICK ONE OF THESE STATEMENTS***, **SIGN THIS FORM** and send it to <Customer_Service_Fax_Number>

☐ We do not have any affected product.

☐ We have found affected product(s): Please confirm the quantity to return above. *If you are returning product not listed above, please **add the UPN, Lot/Batch/Serial number and the quantity to return.***

TO RETURN PRODUCTS:

1. After receipt of the Verification Form, Boston Scientific will contact you to arrange return.

2. Prepare the package

3. Follow the instructions given by your Local Office about collection of the package

NAME* _____ **Title** _____

Telephone _____ Email _____

Customer' **SIGNATURE*** _____ **DATE*** _____

* Required field

dd/mm/yyyy